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Note

Introduction

Please read this manual carefully before using the Product, so as to protect **“the Technical Service Personnel”** and **“the Operator”** from any injury.

Marking

This appliance is a Class 1 medical device pursuant to European Directive on medical devices (MDD) 93/42/EEC (Annex IX) as amended and integrated.

Compliance

The manufacturer declares that this Product complies with Annex I (Essential requirements) of Directive 93/42/EEC as amended and integrated and certifies such conformity by affixing the CE marking.

Validity of manual

This installation manual is valid for the following models:

- ORION TECH, ceiling versions with single and double yoke;
- ORION TECH, floor version.

Customer Service

The customer service is at your disposal in case of Product details, information concerning its use, identification of spare parts being required and for any other queries you might have concerning the appliance, for ordering spares and for matters relating to assistance and warranty.

ordering spares and for matters relating to assistance and warranty.

- TECNO-GAZ S.p.A.
- Strada Cavalli, 4
- I-43038 Sala Baganza PR
- Tel.: ++39 0521 8380
- Fax: ++39 0521 833391
- E-mail: info@tecnogaz.com

Copyright

The reproduction and translation, including partial, of any part of this manual is forbidden without the written permission of TECNO-GAZ.

Translations

The original language of this manual is ITALIAN. For all translations, reference must be made to the original manual language.

Index of contents

KEY	4
1 GENERAL SAFETY INFORMATION.....	5
2 Importance of personal safety.....	5
2.1 Intended use	5
2.2 Safety conditions (secondary effects)	6
2.3 Environmental conditions.....	7
3 General information.....	7
3.1 Operator qualifications	7
3.2 Graphic symbols used in this Operation and Maintenance Manual..	8
3.3 Graphic symbols used on the Product	8
4 Precautions for the Product operator.....	9
4.1 Personnel training obligation	9
4.2 Warranty and liabilities	9
5 Product description and operation	10
5.1 Product description.....	10
5.2 Description of operation	11
5.3 Product handling	12
5.3.1 Brakes for floor version.....	15
5.3.2 Moving the stand.....	15
5.4 Checks to be made every time before use.....	15
6 Cleaning and disinfecting	16
6.1 Application method	16
6.2 Cleaning the Product.....	17
6.3 Product disinfecting	17
6.4 Handpiece sterilization.....	18
7 Adjustment and maintenance.....	19
7.1 Installation of swinging arm	19
7.2 Clutch adjustment	20
7.3 Periodical checks to be performed on the Product	20
7.4 Routine maintenance	21
7.5 Repairs	22

7.6	Disposal after use.....	22
7.7	Spare parts list.....	23
8	Technical properties.....	24
9	Declaration of Conformity of the Manufacturer	26
10	EMC Declaration	27
11	Warranty Certificate.....	33

KEY

PRODUCT	THE EM (Electro-Medical) EQUIPMENT to which this manual refers is a SECONDARY SURGICAL LAMP (TREATMENT LAMP) . For ease of description, in this manual this EM EQUIPMENT will be called “Product” .
OPERATOR	Professional medical personnel (e.g., professional health personnel, expert person assisting the patient).
RESPONSIBLE ORGANIZATION	Entity accountable for the use and maintenance of an EM equipment or EM system (e.g., a hospital, an individual doctor or a non-expert person). Preparation and training are included in use.
TECHNICAL SERVICE PERSONNEL	The personnel (individuals or entity accountable to the responsible organization) that installs, assembles, maintains or repairs the equipment. Under certain circumstances, the safety of such persons depends on their knowledge and training and ability to take appropriate precautions when gaining access to hazardous parts partially. By way of example only, the following professional figures are deemed as SERVICE PERSONNEL: ⇒ Construction Engineer, Draughtsman, Building firm duly registered in the professional Register (for the masonry works) ⇒ Electrical Engineer Electro-technical expert qualified to work as an electrician (for the electrical works)

1 GENERAL SAFETY INFORMATION

This manual is an integral part of the Product as indicated by European Directive 93/42/EEC and subsequent amendments and supplements. Read and keep this Operation and Maintenance Manual close to the Product.

TECNO-GAZ disclaims all liability for any injury to persons or damage to property caused by the USE or MAINTENANCE of the Product by persons who are not OPERATORS or TECHNICAL SERVICE PERSONNEL.

The Product is an EM electro-medical equipment and therefore falls within the field of application of the EN/IEC 62353 standard.

	Electric shock risk
---	----------------------------

To avoid any risk of electric shocks, the Product must only be connected to mains supplies with earth protection.

2 Importance of personal safety

2.1 Intended use

SECONDARY SURGICAL LAMP (TREATMENT LAMP)

The Product is a medical device designed for use in operating theatres within the PATIENT AREA, with short-term duration, active, non invasive, designed to locally light up the patient's body for treatments and diagnosis which can be interrupted without DANGER for the PATIENT in case of a power outage.

A combination of two or more surgical lamps used in the operating theatre and required for treatment and diagnosis makes up a SURGICAL LAMP SYSTEM.

Operating field

The Product correctly lights up the operating range from a distance of about 70 – 140 cm from the patient area.

	Possibility of tissue dehydration and damage
---	---

Undesired effects of overlapping light fields

In the event of overlapping lamps, a temperature increase would ensue in the patient area with consequent risk of dehydration and tissue damage.

In case of a reduction in blood flow with start of tissue dehydration, reduce light intensity.

2.2 Safety conditions (secondary effects)

Optical safety

- Do not direct the light source into the patient's and/or operator's eyes.
 - When Product use is restricted to the face (maxilla-facial surgery, plastic surgery, ear-nose-throat surgery) the patient's eyes must be covered with adequate protection.
- Failure to follow such precautions could cause glare and potential damage to the retina.

Electromagnetic disturbance

To avoid any significant risk of reciprocal interference due to the presence of the Product during specific exams or treatments, refer to section 10 of the Manual.

Incorrect use

Never place and/or hang anything on the Product.

Failure to follow such precaution could result in such objects falling in the operating area.

- Never hang on the Product with the body weight of a person.
- Failure to follow such precaution could damage the Product structure.
- Never cover the head of the Product during operation to prevent overheating.
 - Avoid the Product parts colliding with one another or other nearby equipment.

Knocks could cause the detachment of plastic parts or paint from the Product which could fall in the patient area.

Improper use of floor version

In the case of the floor version, do not rest, push or lie on the product. Failure to comply could result in damage to the product and to devices nearby and injury to staff members.

2.3 Environmental conditions

- The Product is not suitable for use in explosion-risk areas.
- The Product is not suitable for use wherever there are inflammable mixes of anaesthetics with air, oxygen or NO₂ (laughing gas).
- The Product is not suitable for use in environments rich in oxygen and use is not intended in the presence of inflammable agents.
- During operation, the ambient temperature must be between 10°C and 40°C.
- Relative humidity must be between 30% and 75%.
- Atmospheric pressure must be between 700 and 1060hPa.

3 General information

3.1 Operator qualifications

Qualification of personnel in charge of operating on the Product

Use

Professional medical personnel

Cleaning

Properly trained medical and paramedical personnel

Routine maintenance

Qualified technician with required technical-professional skills

Special maintenance

TECNO-GAZ or technical service personnel, the latter only for the fuse change.

Assistance

TECNO-GAZ or authorized Dealer

Demolition

Comply with applicable laws on waste disposal. This product must not be disposed of in standard waste disposal bins. To avoid risks for the environment and health deriving from the dispersion of polluting substances in the environment, separate the various internal component parts such as iron, aluminium, plastic and electrical material, and dispose of these through authorized channels so as to ensure correct recycling.

3.2 Graphic symbols used in this Operation and Maintenance Manual

The following safety measures must be put in place during Product installation, use and servicing.

To emphasize their importance, a number of safety precautions are repeated throughout the manual.

Follow the safety precautions before using or repairing the Product.

Carefully abiding by the safety precautions improves the ability to use the Product safely and correctly and helps prevent incorrect maintenance which could be hazardous and cause damage. The safety measures are approximate and not exhaustive; the Operator, the Responsible Organization and the Technical Service Personnel must develop their capacities to upgrade and integrate them.

General warning signal

General mandatory code of conduct signal

General prohibition signal

3.3 Graphic symbols used on the Product

Below are the symbols to be found on the Product:

CE marking indicating the Product conforms to directive 93/42/EEC and subsequent amendments and supplements

Date of manufacture (month and year)

Manufacturer's address

Fuses used in the device

Comply with the instructions for use

Model

Serial number

Disposal

Protection earth

‘N’

Neutral lead connection point

‘L’

Line lead connection point

‘O’

ON

‘I’


OFF

Standby and switch-on

Pushing, resting on or lying on the product is forbidden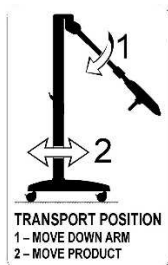

Only move the product after lowering the arm

4 Precautions for the Product operator

4.1 Personnel training obligation

Operator Instructions

The Responsible Organization must instruct the Operator on how to use, clean and service the Product.

The instructions must be provided in written form on the basis of this Manual.

4.2 Warranty and liabilities

TECNO-GAZ disclaims all liability as regards unreliable Product operation in the following cases:

- The Product has not been used for its intended purpose and in conformity with the operating instructions.
- Authorized modifications and repairs have not been performed by TECHNICAL SERVICE PERSONNEL.

5 Product description and operation

5.1 Product description

Versions

The Product is available in various versions:

- **ceiling version with single yoke**
- **ceiling version with double yoke**
- **floor version**

See drawing 193

CEILING version with SINGLE YOKE: ceiling cover (1), ceiling anchor tube (2), supply unit (3), horizontal arm (4), swinging arm (5), yoke (6), lamp head (7), function control keyboard (8), sterilisable grip (9).

See drawing 220

CEILING version with DOUBLE YOKE: ceiling cover (1), ceiling anchor tube (2), supply unit (3), horizontal arm (4), swinging arm (5), external yoke (6), internal yoke (7), function control keyboard (8), lamp head (9), sterilisable grip (10).

See drawing 195

FLOOR version: base with wheels (1), power plug (2), lower stem (3), upper stem (4), swinging arm (5), lamp head (6), function control keyboard (7), sterilisable grip (8).

Separable parts

Sterilisable handpiece Refer to Section 6.4 for assembly/disassembly instructions.

5.2 Description of operation

Main switch

Floor version lamps feature a green light switch for general switching on and off.

In the case of single and double ceiling versions position the thermal magnetic switch near the Product so that it can be switched off in case of need.


CAUTION

Do not position the device (floor version) so it is hard to reach and remove the power plug in case of an emergency.

Control keyboard

The membrane keyboard is located on the yoke. It can control:


- On/Off button (1)
- Sun button (2): light intensity adjustment. The intensity level is indicated by 5 green microleds
- “K” button (3): color temperature selection 4.500 – 5.000K
- “E” button (4): “Endoled” function. Function available only when the lamp is switched off
- Light field diameter selection (5), increase or decrease the light field diameter.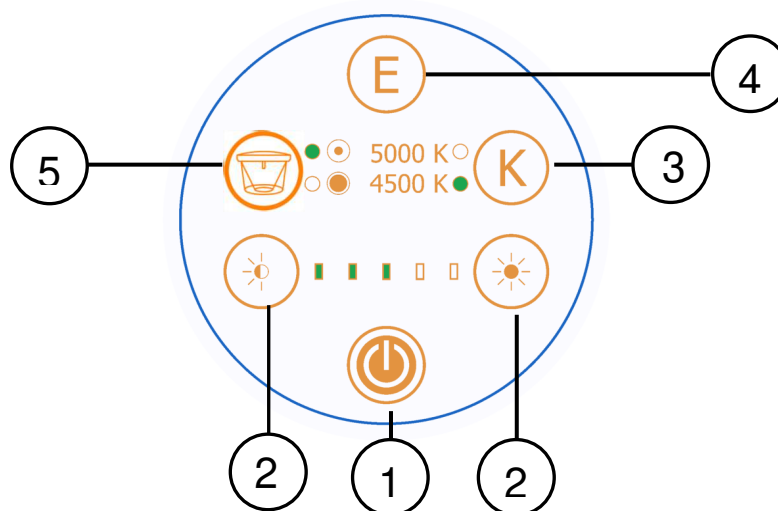



Lighted area

The Product has been designed to be able to regulate the light diameter electronically by means of the key (5) provided.


5.3 Product handling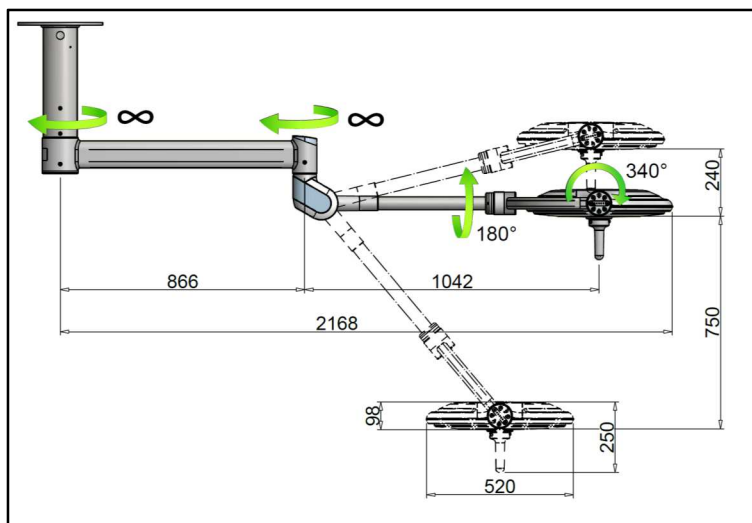


SINGLE ceiling model

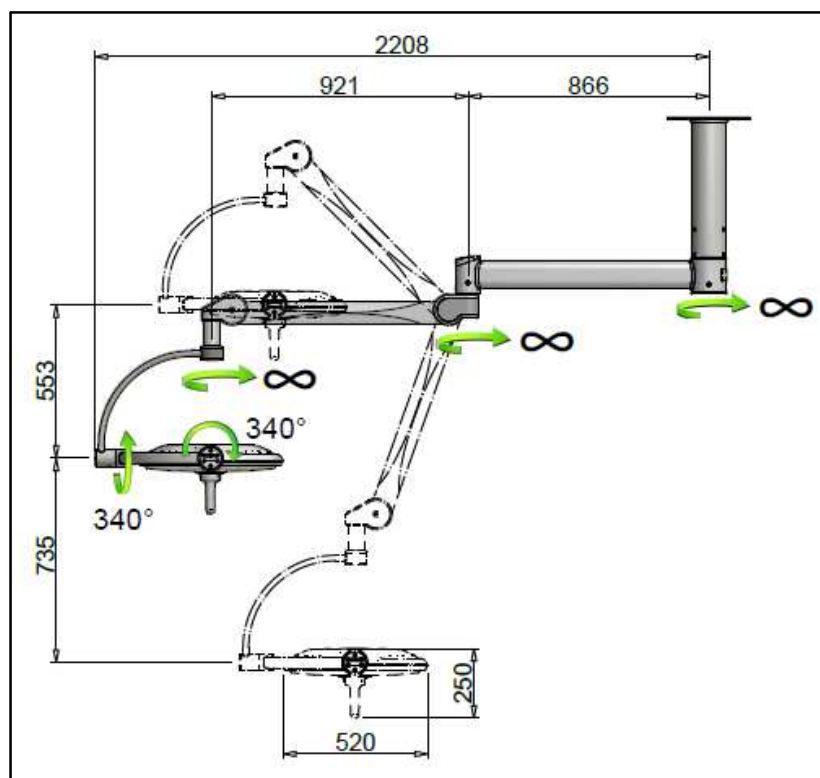
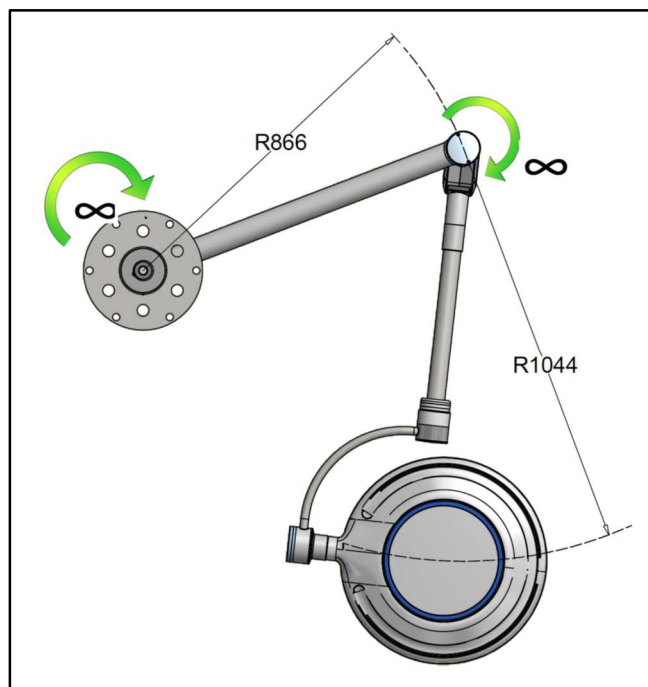
Ceiling model with DOUBLE yoke

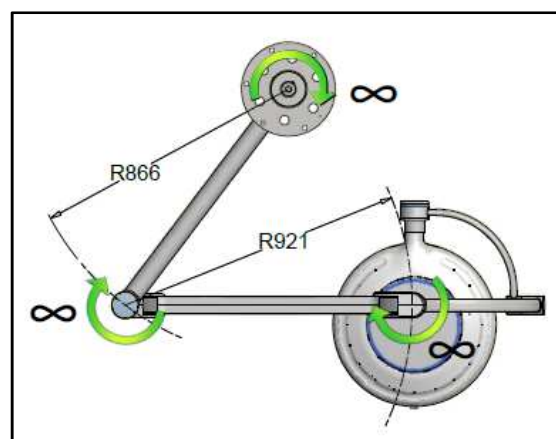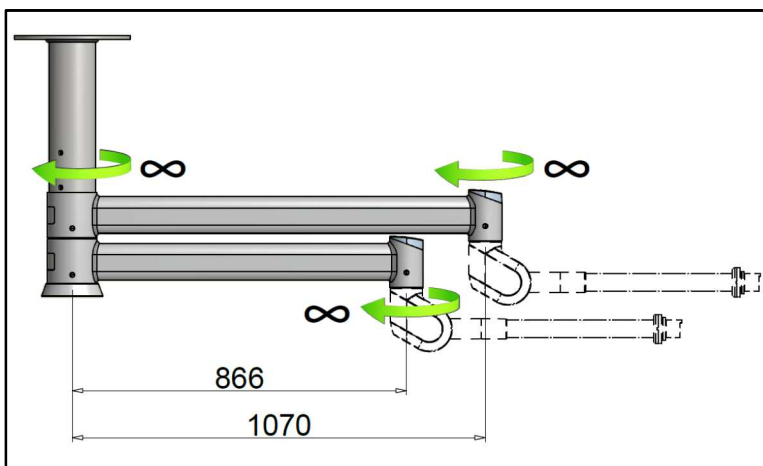
Double lamp model

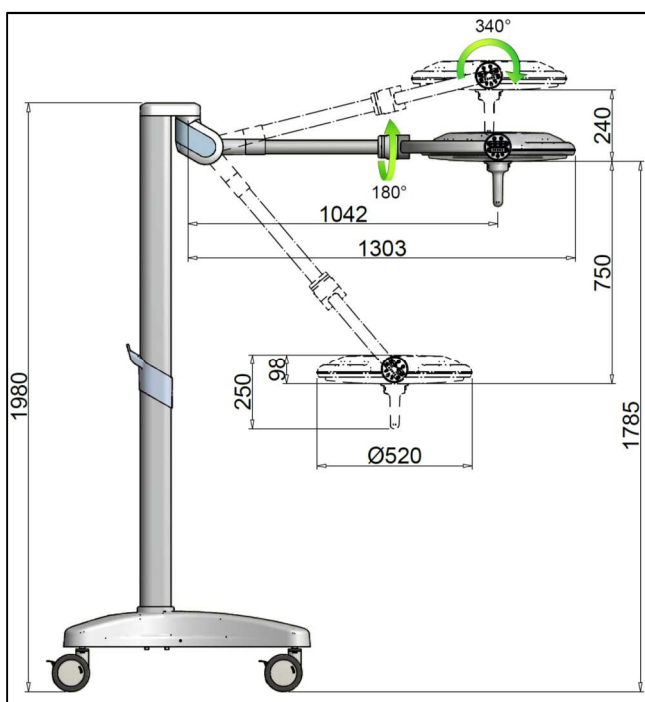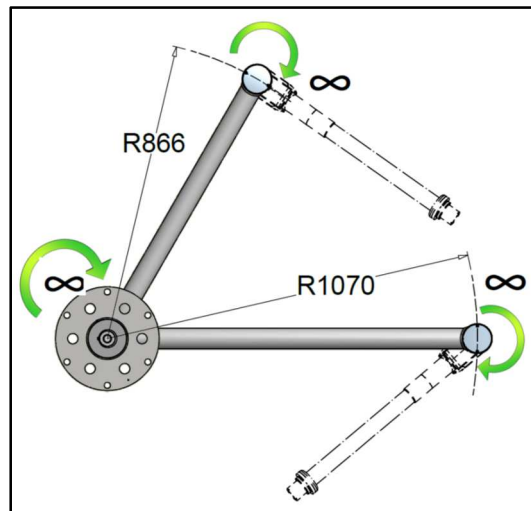Floor model

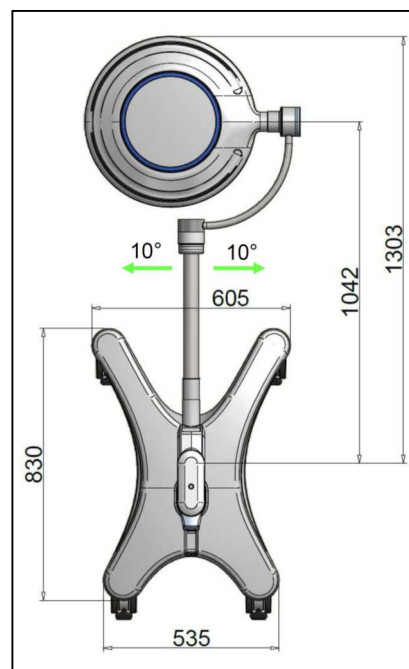The Product can be moved using the sterilisable handpiece (fig. A) or by means of the external contour (fig. B).

Fig.A

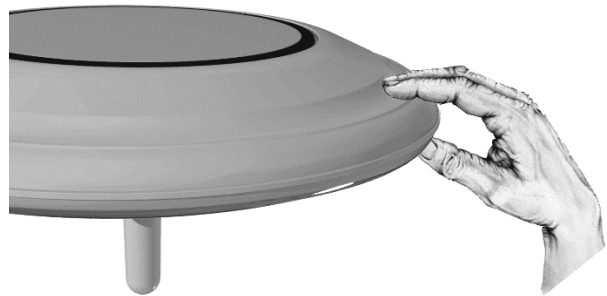
Fig.B

By pressing the keys on the keyboard, the previously described control functions can be enabled (fig.C).



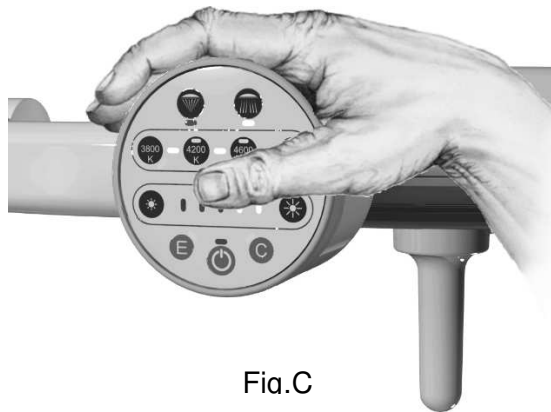
Fig.C

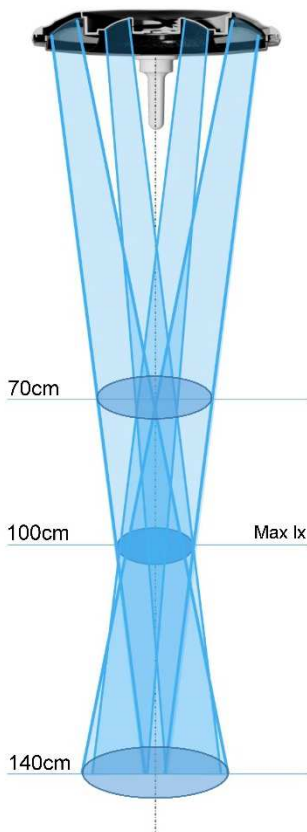RECOMMENDED WORK DISTANCE

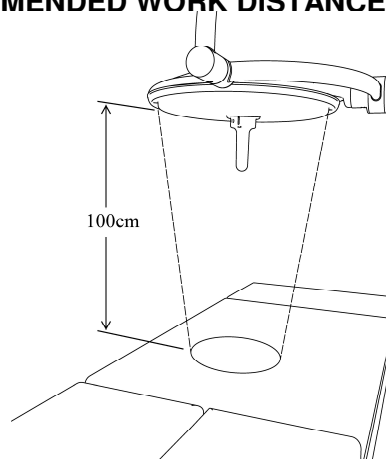To optimize light intensity, the product is best used at a distance of 1 m.

The Product nevertheless also ensures a good light intensity at a distance between 70cm and 140cm.

5.3.1 Brakes for floor version

The floor version has 4 wheels with pedal brake. This are used to block the Product in the required position.

Press the brake pedal with your foot, without applying too much pressure.

Risk of damaging pedal

Do not kick the brake pedal and do not press continuously once the stop position has been reached.

To disengage the brake, lift the pedal with your foot.

5.3.2 Moving the stand

Risk of lamp overturning

Whenever the stand has to be moved, make sure the swinging arm is moved downwards.

Failure to do so could cause the lamp to overturn.

5.4 Checks to be made every time before use

To make sure the Product is safe and provides a correct diagnosis, every time before use, the operator must check:

- The lamp has been correctly disinfected;
- The emitted light is stable and of adequate intensity;
- The swinging arm maintains correctly its position;
- The dome maintains correctly its position.

6 Cleaning and disinfecting

The responsible organization must comply with the rules (standards and directives) concerning hygiene, disinfection and sterilization laid down by the relevant national commission.

6.1 Application method

	Interrupt the power supply before cleaning the Product.
---	--

Before proceeding to clean/disinfect the Product, make sure it is off and cannot be switched back on.


Allow the lamp to cool down and only clean it when it is cold.

Protect the Product from water spray and detergents and do not clean it in direct contact with liquids.

	Possibility of damaging Product
---	--

Application method

Do not spray detergent/disinfectant directly on Product.

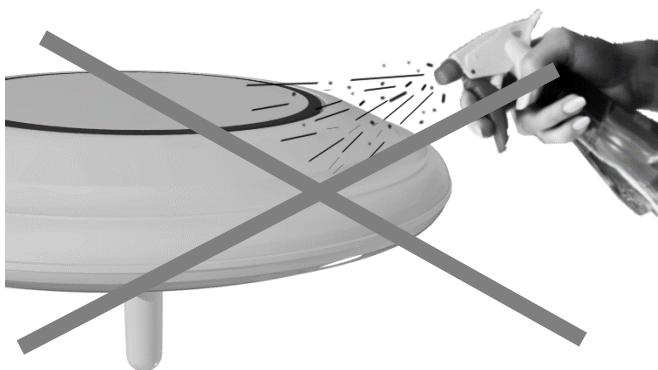Spray the detergent / disinfectant on a cloth so as to dampen it. Afterwards wipe the product with the cloth.

Failure to comply with the above instructions could cause:

- detaching of paint with possible accidental dropping of such paint into the patient area;
- early ageing of the plastic parts with consequent weakening and the possibility of breakages;
- tarnishing of the protection screens and glass.

6.2 Cleaning the Product

Frequency

We recommend you to clean the Product every day.

- Do not use sharp, pointed or abrasive objects, to avoid the risk of damaging surfaces.
- Do not pour liquids directly on the Product.
- Clean the Product with a damp, but not wet, cloth.
- Clean with suitable detergents with low alkaline content and chlorine free. Do not use abrasive products, petrol, paint thinners, alkaline detergents, acids, containing alcohol or aldehydes;
- Dose the detergents strictly according to the percentage indications shown on the manufacturer's technical sheet, being careful that no liquids penetrate into the joints of the various Product parts, with special care give to the reflector and supporting structure.

6.3 Product disinfecting

Frequency

We recommend you to disinfect the Product every time before use.

Disinfectants can contain substances that are harmful for the health; use disinfectants indicated by the national commission for hygiene and disinfection, according to the hygienic standards adopted by the Responsible Organization.

- Do not use sharp, pointed or abrasive objects, to avoid the risk of damaging surfaces.

- Do not pour liquids directly on the Product.
- Disinfect the Product with a damp but not wet cloth.
- Use appropriate disinfectants with low alcohol content.
- To prevent damaging the stainless-steel and aluminium parts, use only disinfectants that do not contain chlorine or halogens.
- Dilute the disinfectants in strict accordance with the percentage indications on the manufacturer's technical data sheet, being careful no liquids penetrate into the joints of the various parts of the Product, with special attention for the reflector and supporting structures.

6.4 Handpiece sterilization

Frequency

The hand-pieces must be sterilized before use and can withstand up to 200 cycles.

The Operator must comply with the rules of the national commission for hygiene, disinfection and sterilization.

The handpieces are made of plastic material resistant to heat and knocks (PSU - Polysulfone).

Replace the hand-pieces as soon as these become cracked or deformed, as these could fall in the patient area.

Hand-piece fitting / removal:

- Press the hand-piece release button and remove it.
- Insert the handpiece in the support, following the guide provided until it is locked in position.

Sterilization

Clean and disinfect the handpieces in the traditional way before sterilization. They can be cleaned with a mid-alkaline detergent free of active chlorine. To disinfect the handpieces, we suggest using alcohol or aldehyde-based products. The disinfectants must be approved by the manufacturer for use on polysulfone (PSU). After disinfecting, rinse off the detergent residues with plenty of water.

The handpieces fit into a suitable sterilization pack (disposable sterilization pack, e.g., plastic/paper bags; single or double pack), before being sterilized.

The handpieces can withstand about 200 steam sterilization cycles in accordance with the following parameters:

- steam sterilization at 121°C 1.3 bar for 25 to 30 minutes
- steam sterilization at 134°C 2.3 bar for 4 minutes

Do not exceed a sterilization temperature of 134°C.

Strictly keep to the ISO 17665-1 standard

When placing in the autoclave, make sure the open side of the handpieces is turned downwards. The handpieces must be free and not burdened by other material being sterilized.

Damaged handpieces must no longer be used.

7 Adjustment and maintenance

7.1 Installation of swinging arm

See drawing 198

The Product is sold already balanced and does not require further adjustment. In the event of the spring swinging arm becoming stiff or loose over time, mechanical intervention is possible by regulating the compression of the internal spring.

Manually move the cover (1) forwards. Fit the adjustment pin (2) in the holes of the ring nut and turn in the direction indicated by the arrows to increase/decrease the load on the spring.

If the swinging arm drops, this means the elastic force of the spring is insufficient:

- turn the lever upwards and release the spring.

If the swinging arm lifts up, this means the elastic force of the spring is too high:

- turn the lever upwards and release the spring.

At the end of adjustment, manually reposition the cover (1) in its original position.

The swinging movement of the arm can also be adjusted upwards.

The Product is sold with maximum set swinging movement. If the swinging movement is to be reduced upwards, manually move the cover (1) forwards and insert the adjustment pin (2) in the second ring nut. By turning the pin downwards, the swinging movement can be reduced until it is in horizontal position.

The swinging movement downwards cannot instead be changed.

At the end of adjustment, manually reposition the cover (1) in its original position.

In the double yoke version for spring adjustment please refer to section 8.4.1, while for the swinging adjustment please refer to section 8.4.2 of ONDAL's "Operating Instructions" manual.

7.2 Clutch adjustment

See drawing 199

The brakes are set during installation. Like all the other mechanical parts, the brakes are also subject to wear.

If the Product does not remain in a stable position, the braking force will have to be adjusted by means of the brake screws.

Use a flathead screwdriver to increase brake force, turning the screws (1) and (2) of the arm brake clockwise.

To increase the braking force at the head, turn the two brake screws (3 and 4) clockwise using a flathead screwdriver.


In the double yoke version, please refer to section 8.4.3 of ONDAL's "Operating Instructions" manual.

7.3 Periodical checks to be performed on the Product

	Making any changes to this device is forbidden.
	Perform the Product electrical check

At the time of start up and after each maintenance job, perform electrical tests and jobs indicated in the IEC 62353 standard.

7.4 Routine maintenance

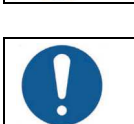	Interrupt the power supply before doing any maintenance jobs.
	Check Product integrity

N	Period	Action
1	Before using	Make sure there are no pieces or fragments of paint that could become detached and fall within the operating field. If there are any, remove them manually.
2	Before using	Make sure the light source protection screens are not damaged. If they are, contact the Customer Service.
3	Once a year	Check all the Product joints and make sure there are no noises or squeaks. If there are, lubricate the clutches involved with suitable grease for industrial use at a service temperature between -30°C and + 120°C, type OKS 470 or with similar properties.
4	Once a year	If the Product fails to maintain a regular position, adjust the clutches as indicated at points 7.1 and 7.2 (arm and clutch adjustment) .
5	Once a year (CEILING VERSION)	Make sure the bar retention screws (1) are tightened properly. Also check the bar horizontal arm retention screws (4). If these are not properly fastened, adequately tighten. See drawing 200. To access the nuts (1), slide off the aluminium ring (2) downwards by loosening the screws and bar cover (3).
6	Once a year (FLOOR VERSION)	See drawing 200. Floor version: Make sure the stem retention screws (5) under the base are tightened properly. If these are not properly fastened, adequately tighten.

7.5 Repairs

	The Product must only be opened and repaired by the Technical Service Personnel for the fuse change. All other repairs to be done by the manufacturer.
	Interrupt the power supply before doing any maintenance jobs.
	Making any changes to this device is forbidden.

The only repair job with which the technical assistance personnel are charged is the fuse change.

See drawing 200

To access the fuses in the ceiling version, open the bar cover as indicated in point 5 of paragraph 7.4.

See drawing 201

To access the fuses in the floor version, lift the ring (7), remove the screws (8) and lift the cover (9).

Remove the screw fuse carrier (5) from the terminal board and replace the fuse (6) making sure it is replaced with another of the same type.

If necessary, Tecno-gaz will provide the wiring diagrams, the list of component parts, the descriptions, setting instructions or other information to assist the technical assistance personnel in repairing the parts of the Product indicated as reparable by the technical assistance personnel.

If the above indications are not enough to solve the problem, contact the after-sales service.

7.6 Disposal after use

Disposal at end-of-life

Comply with applicable laws on waste disposal. This product must not be disposed of in standard waste disposal bins. To avoid risks for the environment and health deriving from the dispersion of polluting substances in the environment, separate the various internal component parts such as iron, aluminium, plastic and electrical material, and dispose of these through authorized channels so as to ensure correct recycling.

7.7 Spare parts list

	Only original spare parts must be used
---	---

Description	Order code
Sterilisable grip	Z400307-2
Electronic board	Z300632-U29DS
Keyboard	Z300227
0/1 switch (for floor versions)	Z300016
Switching power supply unit	Z170180
Fuse T2AH 250V '5x20'	Z400195
Fuse T4AH 250V '5x20'	Z400215
Fuse T6.3AH 250V '5x20'	Z400216

8 Technical properties

LAMP MODEL:		ORION TECH	
TECHNICAL DETAILS		VALUE	
Illumination E _c at 1 m ± 10% [Lux]		130,000	
Illumination E _c at 0.7 m ± 10% [Lux] (DENTAL CARE)		70,000	
Colour temperature [K]		4,500	5,000
Colour rendering index R _a [-]		96	
R ₉ [-]		≥ 90	
Light field diameter d ₅₀ [mm]		95	
Light field diameter d ₁₀ [mm]		190	
Light field diameter [mm]		270	
Light field diameter [mm] (DENTAL CARE)		200 x 150	
Lighting depth L1+L2 [cm] (60%)		27 + 13	
Lighting depth L1+L2 [cm] (20%)		55 + 40	
Primary alternate voltage [Volt ac]		100 - 240	
Frequency [Hz]		50 / 60	
Absorbed power [VA]		60	
Light source		29 LED	
Duration of LED diode light source [hr] (this figure can vary according to power peaks and operating frequency)		> 60,000	
Light intensity control [%]		20 - 100	20 - 100
Colour		RAL 9003	
Directive		93/42/EEC and 2007/47/EC	
Standards		EN/IEC 60601-2-41	
Classification of Medical Device		Class I	
Essential performance	Distribution of minimum and adequate lighting (luminous flux emitted by the EM equipment shall not vary by more than 20% during use; the colour temperature and colour rendering index shall be stable and within the range 3000K-6700K and 85-100, respectively; E _c value shall be ≥ 40,000 lux and ≤ 160,000 lux).		
	Limitation of energy in the operating field (UV-irradiance for wavelengths below 400 nm shall not exceed 10 W/m ² ; and the total irradiance E _e in the lighted area shall not exceed 1000 W/m ² at a distance of 1000 mm; Ec value shall be ≥ 40,000 lux and ≤ 160,000 lux; E _e /E _c ≤ 6 mV/m ² lx).		


IP degree of protection	IP20
Operating conditions	Continuous operation
Handpiece steam sterilization	121°C 1.3bar for 25 to 30 minutes 134°C 2.3bar for 4 minutes
Mains power voltage insulation means	Outside the product (main switch) for ceiling versions Main switch for floor versions
Diameter of lamp body [cm]	52
Weight of surgical lamp [kg]	35
CE	In conformity with Directive 93/42/EEC (and 2007/47/EC)

9 Declaration of Conformity of the Manufacturer

The company:

TECNO-GAZ S.p.A. Strada Cavalli n. 4 - CAP 43038 - Sala Baganza - Parma - ITALY

declares under its own responsibility that the Product (Medical lighting device for surgical and diagnosis use):

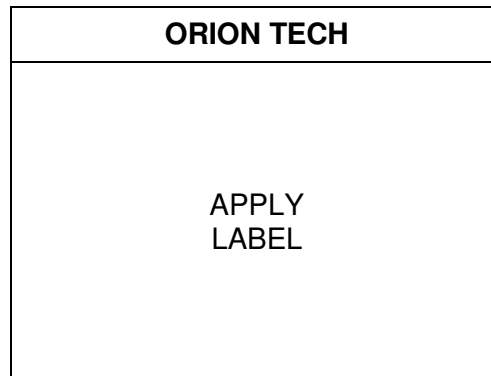made by TECNO-GAZ S.p.A, complies with Annex VII of Directive 93/42/EEC dated 14/05/1993, enforced in Italy by Legislative Decree No. 46 dated 24 February 1997 and subsequent amendments (including Directive 2007/47/EC dated 05/09/2007, enforced in Italy by Legislative Decree No. 37 dated 25 January 2010) and with the following standards:

- EN/IEC 60601-1 (Part 1: General requirements for basic safety and essential performance)
- EN/IEC 60601-2-41 (Part 1: Particular requirements for the safety of surgical luminaires and luminaires for diagnosis)
- EN/IEC 60601-1-2 (Part 2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests)

Classification with reference to article 9 and Annex IX of Directives 93/42/EEC and 2007/47/EC

DURATION:	Short term (Annex IX, Par.1 "Definitions", art.1, subsection 1.1)
DESCRIPTION:	DESCRIPTION: Non-invasive medical device (Annex IX, Par.1 "Definitions", art.1, subsection 1.2) DESCRIPTION: Active medical device (Annex IX, Par.1 "Definitions", art.1, subsection 1.4)
CLASS I:	(Annex IX, Par.3 "Classification", art.3, subsection 3.3, Rule 12) and (Annex IX Par.3 "Classification", art.1, subsection 1.1 Rule 1)

- The conformity assessment is developed with reference to article 11 of Directive 93/42/EEC and 2007/47/EC.
- The TECNO-GAZ Quality System complies with UNI EN ISO 9001 and UNI CEI EN ISO 13485 standards and is certified by CSQ (CSQ certificate no. 9120.RMS1 and 9124.RMS2).

Name: PAOLO BERTOZZI
Position: Managing Director

10 EMC Declaration

The Product has been tested according to EN60601-1-2 standard to ensure correct electromagnetic compatibility.

Portable and mobile communication appliances can affect the Product. The product should not be used close to another device and if this is inevitable, the product must be checked to make sure it is working properly.

The use of accessories other than those supplied/recommended by the manufacturer could increase the level of emissions and lower the level of immunity of the appliance.

The Product has been designed to be used in the electromagnetic environments described below.

The Responsible Organization or Operator is responsible for making sure the Product is used in a compatible environment.

Possibility of interferences with nearby appliances

It could occur that if the Product is affected by radiations in the range of 80 MHz – 1 GHz or bursts, it will no longer respond to the commands both as regards the lamp and the camera.

If this does occur, essential performance will in any case be ensured, but to restore normal operation it will be necessary to de-energize the master switch.

Possibility of interferences with nearby appliances

Immunity test	Compliance	Electromagnetic environment - directives
RF Emissions CISPR 11	Group 1	The Product only uses RF energy for internal operation. Consequently its RF emissions are very low and should not cause any interference to nearby electronic appliances.
RF Emissions CISPR 11	Class A	The Product is suitable for use in all environments except in domestic environments and those directly connected to a low-voltage public mains supply which supplies buildings used for domestic purposes, as long as the following precaution is followed. Warning: This Product is intended for use by professional health personnel only. This Product can cause radio-interference or disturb the operation of nearby appliances. Measures may have to be taken to reduce such disturbance, such as Product re-positioning or shielding of premises.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Conforming	

Immunity test	Test level to EN/IEC 60601-1-2	Conformity level	Electromagnetic environment - directives
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 8 kV at contact +/- 15 kV in air	+/- 8 kV at contact +/- 15 kV in air	Floors must be made of wood, concrete or ceramic tiles. If the floors are covered with synthetic material, relative humidity must at least be equal to 30%.
Rapid impulse electric transients IEC 61000-4-4	+/- 2 kV For electric power lines +/- 1 kV For input/output lines	+/- 2 kV For electric power lines +/- 1 kV For input/output lines	Mains voltage quality should be that of a typical commercial or hospital environment.
Overvoltage IEC 61000-4-5	+/- 1 kV Between phases +/- 2 kV Between phases and earth	+/- 1 kV Between phases +/- 2 kV Between phases and earth	Mains voltage quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and variations on the power supply input lines IEC 61000-4-11	<5% U_T (drop >95% of U_T) For 0.5 cycles <40% U_T (drop = 60% of U_T) For 5 cycles <70% U_T (drop = 30% of U_T) For 25 cycles <5% U_T (drop >95% of U_T) For 5 s	<5% U_T (drop >95% of U_T) For 0.5 cycles <40% U_T (drop = 60% of U_T) For 5 cycles <70% U_T (drop = 30% of U_T) For 25 cycles <5% U_T (drop >95% of U_T) For 5 s	Mains voltage quality should be that of a typical commercial or hospital environment. If the Product user requires continued function during mains power supply interruptions, the Product should be supplied by a UPS unit or batteries.
Magnetic field at electrical mains frequency (50/60Hz) IEC 61000-4-8	30 A/m	30 A/m	The magnetic fields at mains frequency should have the characteristic levels of a typical locality in a commercial or hospital environment.
NOTE U_T mains voltage in AC before application of test level.			

Immunity test	Test level to EN/IEC 60601-1-2	Conformity level	Electromagnetic environment - directives
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 Veff 150 kHz to 80 MHz 3 V/m 80 MHz to 2.7 GHz	3 Veff 3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the Products, included cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1,2\sqrt{P} \quad 150 \text{ KHz to } 80 \text{ MHz}$ $d = 1,2\sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2,3\sqrt{P} \quad 800 \text{ MHz to } 2.7 \text{ GHz}$ where P is the maximum output power rating of the transmitter in watts (W), according to the transmitter manufacture and d is the recommended separation distance in meters (m). Field strengths from fixed transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Test frequency (MHz)	Band ^{a)} (MHz)	Service ^{a)}	Modulation ^{b)}	Maximum power (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
385	380-390	TETRA 400	Pulse modulation ^{b)} 18 Hz	1.8	0.3	27
450	430-470	GMRS 460, FRS 460	FM ^{c)} ± 5kHz deviation 1 kHz sine	2	0.3	28
710	704-787	LTE Band 13, 17	Pulse modulation ^{b)} 217 Hz	0.2	0.3	9
745						
780						
810	800-960	GSM800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation ^{b)} 18 Hz	2	0.3	28
870						
930						
1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation ^{b)} 217 Hz	2	0.3	28
1845						
1970						
2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation ^{b)} 217 Hz	2	0.3	28
5240	5100-5800	WLAN 802-11 a/n	Pulse modulation ^{b)} 217 Hz	0.2	0.3	9
5500						
5785						

NOTE if necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1m test distance is permitted by IEC 61000-4-3.

- ^{a)} For some services, only the uplink frequencies are include.
- ^{b)} The carrier shall be modulated using a 50% duty cycle square wave signal.
- ^{c)} As an alternative to FM modulation, 50% pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

Recommended separation distance between portable an mobile RF communications equipment and the Product

The Product is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Product can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Product as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2.7 GHz $d = 2,3\sqrt{P}$
0.01	0.12	0.12	0.24
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: at 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects an people.

11 Warranty Certificate

1. The Product is covered by an 18-month warranty, including electrical parts
2. The warranty begins on the date of Product shipment from the TECNO-GAZ warehouse to the buyer.
3. In case of disputes, the date indicated on the “transport document” attached to the goods shall be deemed valid.
4. The warranty only covers the sending of Product spare parts to the buyer or, in the event of TECNO-GAZ considering the replacement of spare parts not feasible, the replacement of the entire product, after fabrication faults have been properly ascertained at the undisputable judgement of TECNO-GAZ. The warranty does not therefore cover any other costs or expenses (including, by way of example but without limitation, labour costs, packaging costs and transport costs, etc.).
5. The guarantee does not include the components subject to normal wear, such as halogen bulbs, LEDs, fuses, relays, ball bearings, etc.)
6. The warranty does not cover:
 - malfunctions due to failure to comply with all instruction manuals;
 - malfunctions due to installation and/or maintenance errors;
 - malfunctions or faults caused by carelessness, negligence, incorrect use or other causes not attributable to TECNO-GAZ;
 - malfunctions or faults due to the fact that the electrical system of the premises where the device is installed is not in compliance with CEI 64-8 standards (standards for electrical systems in premises used for medical purposes) and similar standards.
7. TECNO-GAZ shall repay direct damages suffered by the buyer and which are documented as attributable to its product, caused within the warranty period, for an amount not above 40% of the net value of the product as indicated on the buyer’s invoice. TECNO-GAZ’s liability is expressly ruled out for indirect damages or consequential damages (including cases of the Product not being used) deriving from the supply.
8. This warranty certificate replaces legal warranties for faults and non-conformities and rules out any other possible liability of TECNO-GAZ originating from the supplied products.
9. The payment of any damages to persons or things due to product malfunction or faults shall be limited to the maximum amount of TECNO-GAZ’s insurance coverage for civil liability.
10. The warranty shall be automatically invalidated in the event of:
 - the Product having been tampered with or modified by the buyer or third parties;

- the Product having been repaired by the buyer or third parties, without following the instructions in the instruction manuals;
- the Product serial number having been cancelled, defaced or removed;
- the buyer not being up to date with payments.

11. For jobs to be done under warranty, the buyer shall contact TECNO-GAZ only.
12. The component parts replaced under warranty must only be returned to TECNO-GAZ, if so requested by TECNO-GAZ, carriage free and suitably packed.
13. In case of failure to return a part requested by TECNO-GAZ, the cost of the component part will be charged.
14. TECNO-GAZ cannot accept returns from end users or in any case from parties other than the buyer.
15. Products returned to TECNO-GAZ must be complete with documentation authorising such return and another document describing the malfunction.
16. For everything not indicated on this warranty certificate, reference shall be made to the laws of Italy
17. For all disputes deriving from or related to the orders to which this warranty certificate applies and which cannot be amicably settled between the parties, the only competent law court shall be that of Milan.

	Operation and maintenance manual	MU_029_EN	07/11/18	EN
		Rev.0	Page 35 of 35	

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